

Cleaning Manifold



Compatible with software version 2.5.0 and newer

IntelliPlex 1000

Cleaning Manifold_P-7_V02

- Make sure V tray is removed from SA-PE tank



V Tray (SA-PE Tank)



IntelliPlex 1000

Cleaning Manifold_P-7_V02


- Select **Maintenance** under Setting menu
- Select **Clean Manifold**
- Click **OK**

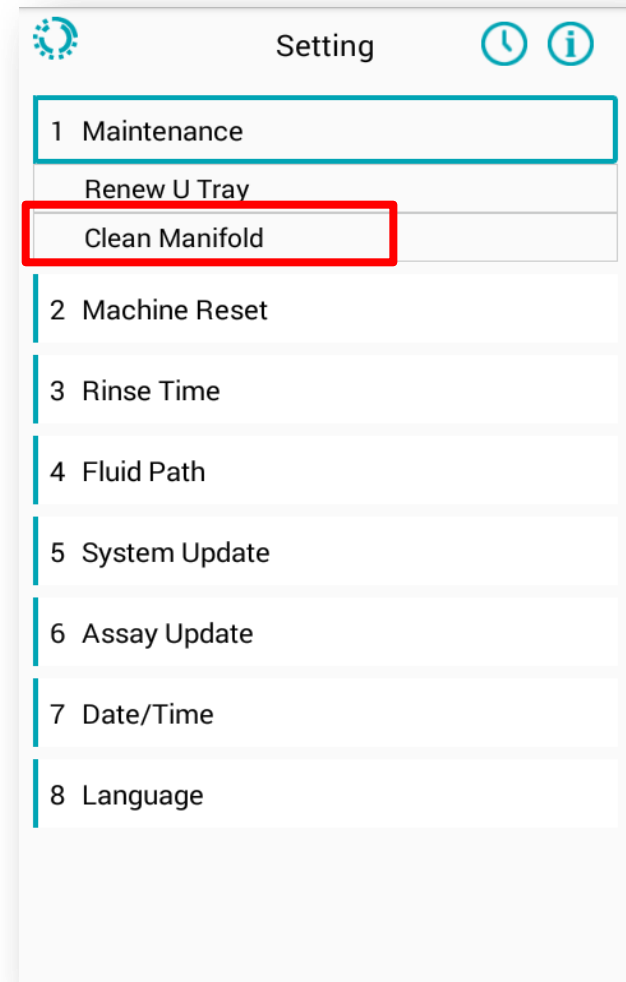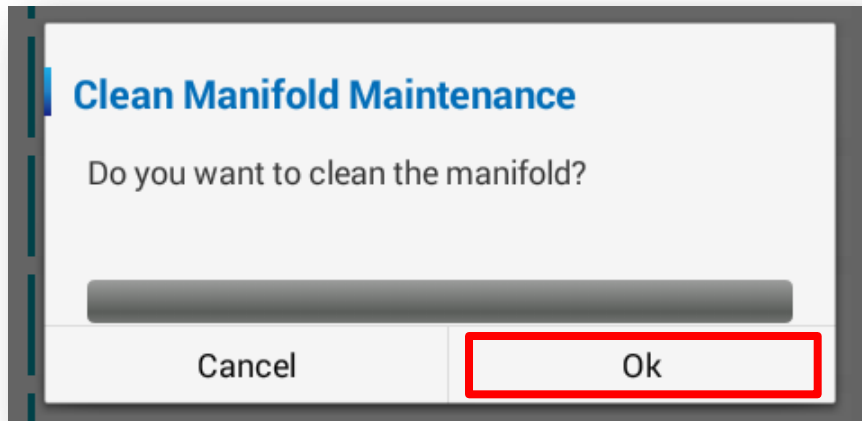

IntelliPlex 1000

Cleaning Manifold_P-7_V02

- The manifold will move to the front
- Use needles in the accessory box to clean clogged manifold

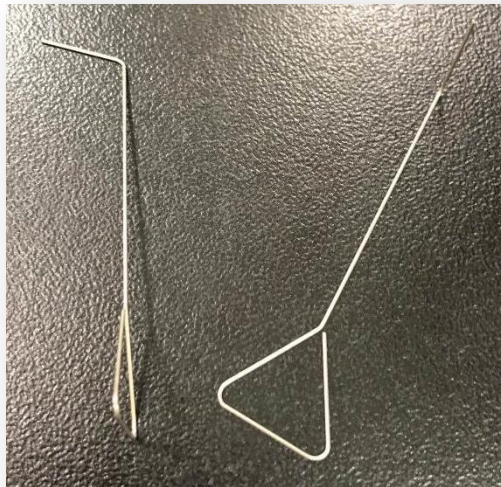

Thank You

PlexBio

6F-1, No. 351, Yangguang St.
Neihu District, Taipei City 11491, Taiwan
+866-2-2627-5878 | PlexBio.com
service@plexbio.com